

SecureVac™ Immobilization System

Cost-effective and easy to use system for all your treatment needs.

New SecureVac™ vacuum cushions by Bionix are durable, accurate and cost-effective. They are reusable and airtight filled with small polystyrene spheres to immobilize the patient. SecureVac cushions feature a new nylon material that is stronger to prevent punctures and leaks. The SecureVac cushion is easy to use, just place patient in desired position and evacuate the air with SecureVac pump. If you need to reposition, inflate and adjust the polystyrene balls and evacuate air until the cushion becomes rigid.

Durability

The cushions are made of durable nylon so the material is thicker to give you a more accurate and rigid mold. The durable nylon also helps prevent punctures and leaks, which provides a longer lifetime. It also features a double clamp system for more security.

Versatility

The cushions can be used in most treatment fields for versatile treatment styles. SecureVac cushions are adaptable to industry standard pumps. The SecureVac can also lock down to your Varian Exact™ Couch with an adapter. The SecureVac comes in many different sizes and fills to meet all your treatment needs.

Easy to Use

Adjust patient on to cushion in desired position and evacuate air. Positions can be easily corrected during simulation in a matter of seconds by adding air to the cushion. The cushion is easy to close off by pinching the closure clamps with one hand. The SecureVac cushion has a grommet in the corner for easy storage.

To order, or for more information call **Bionix Radiation Therapy** at 1.800.624.6649 or visit us on the web at www.BionixRT.com.



The SecureVac Cushion with window



RT-SecureVac



The SecureVac comes in many different sizes and fills to meet all your treatment needs.

SecureVac™ Sizes

- #7815 - 70 x 70cm, 15 Liter fill
- #7820 - 70 x 70cm, 20 Liter fill
- #7530 - 100 x 70cm, 30 Liter fill
- #7535 - 100 x 70cm, 35 Liter fill
- #7450 - 100 x 100cm, 50 Liter fill
- #7380 - 100 x 150cm, 80 Liter fill

Note: Call Bionix for other available sizes.

